

## LUXOFOAM

### A. GENERAL PRODUCT INFORMATION

LUXOFOAM is a self-adhesive soft PVC foam with closed cells. It is used as a softening material on orthoses, external immobilization devices and rehabilitation aids.

### B. PRODUCT RANGE

LUXOFOAM is available in sheets of 50 cm (width) x 97 cm (length) x 3 mm (thickness).

Art. no.	Perforation type
35110	non-perforated
35112	perforated

### C. PRECAUTIONS BEFORE USE

- \* Cut the required pattern before removing the paper from the back.
- \* Stick LUXOFOAM on the orthosis either before activating the orthotic fabrication material or after making the orthosis. In the latter case, the extra space which is taken by LUXOFOAM should be taken into account.
- \* LUXOFOAM can still be easily removed immediately after application. After a while, this will be difficult, unless the orthosis is reheated.

### D. MAINTENANCE AND WASTE MANAGEMENT

LUXOFOAM must be cleaned daily. Use lukewarm water and disinfecting soap or pre-moistened isopropanol wipes. Rinse well.

#### ! Never use solvents.

Sterilization of low temperature thermoplastics with LUXOFOAM in an autoclave is impossible. Sterilization by means of gas treatment is possible. Disinfection is possible with alcohol, quarternary ammonium or a solution or commercial disinfection soaps (HAC®, Sterilium®, etc.).

After use, the orthosis can be disposed of with normal household waste without harming the environment.

### E. ADDITIONAL INFORMATION

For additional information such as product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 31300  
VERSION 5  
LAST UPDATE: 25/05/2021  
REVISION DATE: 25/05/2023