

## ORFILASTIC

### A. GENERAL PRODUCT INFORMATION

ORFILASTIC is an elastic terrycloth lining material with a backing of self-adhesive open cell foam. It is used to improve air transmission of orthoses, external immobilization devices and rehabilitation aids and to ameliorate the comfort of wearing these products.

### B. PRODUCT RANGE

Art. No.	Colour	Width: 50 cm wide Length: 1 m Thickness: 3 mm
35111	skin	
35117	black	

### C. PRECAUTIONS BEFORE USE

- \* Cut the pattern before removing the release paper from the back (direction has no importance).
- \* ORFILASTIC is fixed on the inside of the orthosis, after the orthosis has been formed.
- \* Remove the release paper and stick ORFILASTIC on the required area.
- \* Proceed with removing the paper while sticking ORFILASTIC in the orthosis under slight pressure. The pressure must avoid the emerging of folds.
- \* Make sure that ORFILASTIC does not stick back to back: this is irrevocable.

### D. MAINTENANCE AND WASTE MANAGEMENT

- \* ORFILASTIC is NOT waterproof and has to be replaced when necessary.
- \* After use, ORFILASTIC can be disposed of with normal household waste without harming the environment.

### E. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

**Note:**

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. No. 31301  
VERSION 6  
LAST UPDATE: 25/05/2021  
REVISION DATE: 25/05/2023